

Hand Bulb with Pressure Gauge

adjusts PneumaticWalker™ aircell compression (01PG)

To inflate aircells, firmly insert “inflate” tip of hand bulb into valve and squeeze bulb until the **bottom edge** of the pressure gauge indicator is above the blue stripes (over 30). With bulb still in valve, stop squeezing and pressure gauge indicator will slowly fall. Remove hand bulb from valve when the **bottom edge** of the pressure gauge indicator falls within desired section of pressure guide (see *Close-up of Pressure Guide*).




Inflate aircells in order:

1 - Anterior
(front panel)

2 - Medial Distal

3 - Lateral Distal

4 - Proximal

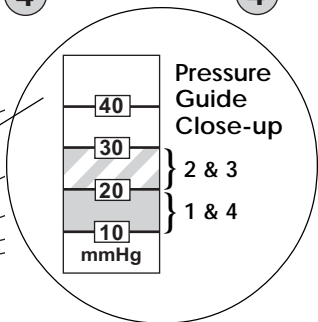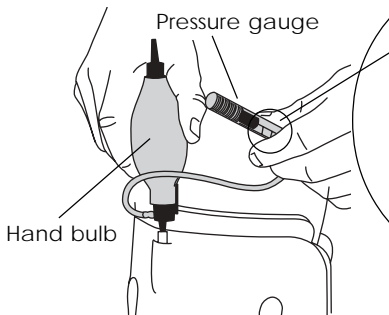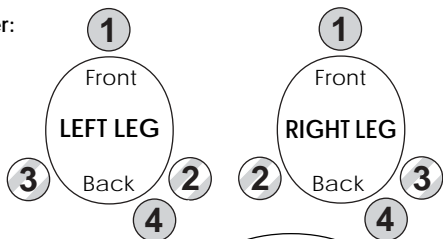

Note: To deflate, firmly insert “deflate” tip of hand bulb into valve and squeeze bulb. Aircell compression should be checked with the pressure gauge whenever brace is adjusted and/or straps are tightened or loosened.

AIRCAST
INCORPORATED

Summit, NJ/USA 07902-0709

(908) 273-6349

800-526-8785

www.aircast.com

CE
0123

R 12/17/99
01P139F
0137